

TS 109 – GTC SUPPLEMENT

INTENDED USE

For selective isolation of Enterococci.

COMPOSITION

Ingredients	Concentration
Gentamicin	1.250mg

(per vial sufficient for 500 ml medium)

Applied in Medium



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





TM 1190 - GTC AGAR BASE

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml sterile distilled water and aseptically add to 500 ml sterile, molten, cooled (45-50°C) TM 1190 - GTC AGAR BASE along with 10 ml of sterile 10% Sodium bicarbonate solution just before use. Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Mallat Gold Bachstraße 10, 40142 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

***For Lab Use Only**
Revision: 28 Feb., 2022